

DISPENSED BY YOUR PHYSICIAN

REVITADERM[®]

WOUND CARE GEL



Patient Instructions*

1. Consult your physician before use and always follow his or her instructions
2. Clean wound before each application
3. Apply a thin layer of RevitaDERM[®] Wound Care Gel to the wound surface and skin immediately surrounding the wound
4. Cover wound as directed by your physician

*Complete Instructions on Product Label

FAQs:

Q: Is it safe to go out into the sun without gauze or a bandage after application?

A: No, if you are going to expose the wound to the sun for a prolonged period of time, the wound should be covered. Consult your physician for additional guidance.

Q: After a few days, the wound appears deeper, is this normal?

A: Yes, RevitaDERM[®] Wound Care Gel is formulated to clear damaged tissue from the wound.

Ask Your Physician About

SCARCARE[™]
GEL - PAD KIT



Blaine Labs, Inc., Santa Fe Springs, CA 90670
Contact@BlaineLabs.com (800) 307-8818
Visit Us At www.BlaineLabs.com